NO MORE CORDS
BETWEEN A MONITOR
AND A SENSOR MAT

MULTI PURPOSE
ALARM MONITOR,
POWERED ADAPTOR
OR BATTERY

WIRELESS TRANSMITTER INCLUDED (RJ11)

REQUIRES BED OR
CHAIR MAT WITH RJ11
CONNECTION

USED WITH A NURSE CALL SYSTEM IF WIRED FOR TIP & SLEEVE ONLY

USED STAND-A-LONE
IN A COMMMUNITY
AREA

REDUCE THE RISK OF FALLS & POTENTIAL INJURIES

PORTABLE

EASY TO USE

PLUG & PLAY OPERATION

COST EFFECTIVE



DELUXE FALL ALARM MONITOR & WIRELESS TRANSMITTER KIT





PRODUCT CODE:

MED-TS2023K-W6T3 – DELUXE FALL ALARM MONITOR & WIRELESS TRANSMITTER KIT Kit includes: Monitor, Nurse Call Connection Lead – Tip/Sleeve (T3), Wall Mount Bracket, Power Pack, Silicon Strap, and Transmitter

Indigo Care's 8 Channel Falls Monitor & Transmitter is designed to be used in conjunction with our chair & bed mats/pads that have a RJ11 connection. No more cords between a monitor and a sensor mat! Our unique kit is multifunctional and can be corded or cordless, with the capability of being used with up to 8 sensor mats.

Indigo Care's bed or chair mat's RJ11 connection will plug into the Fall Monitor's Transmitter which is powered by $2 \times AAA$ batteries. Connect your sensor mat to the Transmitter's RJ socket and switch the on/off switch to the ON position to program to the Fall Monitor. (separate programming instruction are provided).

Once the transmitter is paired with the Monitor and while using the included Nurse Call Connection Lead (Tip & Sleeve only), connect the Monitor to your Nurse Call Point. We have included a wall mount bracket which allows easy installation of the Monitor beside your Nurse Call Point. The Monitor is powered either by its included power adaptor or 3 x AA batteries that you can purchase separately. Three different alarm tones or mute mode can be selected as required. Once a resident gets up from the sensor mat, it will alarm and alert the staff, through your nurse call system. That's it; easy to set up and easy to use!

A unique feature of the Indigo Care Monitor is its ability to program up to 8 sensor mats and transmitters giving you more monitoring options for use in a stand-a-lone situation. It can be easily used with a wheelchair or in a communal area, from 1 to 8 chair sensor mats per Monitor. This will allow your residents to be monitored anywhere, no matter the location

Note: Monitor & Transmitter are sold as a Kit. Sensor Mats can be purchased separately.

KEY SPECIFICATIONS:

- Nurse Call, DC Output & Battery Compartment
- Program up to 8 wireless Bed or Chair Mats RJ11 connection
- Alarming Monitor Visual Screen
- 1/2 metre cable with 6.3mm Stereo Jack Plug T3 (Tip/Sleeve)
- 3 Selectable Alarm tones
- 3 levels Volume Control: Low, Medium or High
- 3 Delay Settings: 0 sec, 2 sec or 15 min
- Reset/Pause Button Pause up to 10 minutes, cancel at any time
- · Volume Control and Mute Button
- Lower Battery Warning
- · Plug Pack Disconnect Warning
- Transmitter Offline Warning
- Power On/Off Switch
- Monitor 12V Power Adaptor (included) or 3 x AA Batteries (not included)
- Transmitter 2 x AAA Batteries (not included)
- · Warranty: 1 year

INDIGO CARE

(Horizon Electronics Company, T/as) 2/93 Rushdale Street, Knoxfield, VIC, 3180 PH +61 3 9763-2988 www.indigocare.com.au ABN 36007302801

ORDER INFORMATION
Email your purchase orders to sales@indigocare.com.au