

DELUXE FALL ALARM MONITOR & WIRELESS TRANSMITTER KIT – INSTRUCTION MANUAL

SET UP INSTRUCTIONS

Our wireless kits are pre-programmed.

To set up your kit:

- Place batteries in monitor compartment and/or connect the power pack.
- Connect transmitter to mats/pads and switch transmitter to the on position.
- If required, Connect the monitor into your call point (tip & sleeve compatible only)
- Then trigger your wireless device to test.

How to trigger your wireless devices:

- Bed/Chair/Under Mattress Mats and Pads: move weight off the mat or pad

INDIGO CARE PRODUCT CODES

CODE	DESCRIPTION
MED-TS2023-W6T3	Deluxe Alarm Fall Monitor/Transmitter Kit
MED-TS2023-TX	Wireless Transmitter (for use with Fall Monitor only)
MED-C365-NC10	Chair Sensor Pad – RJ11
MED-B365-NC20	Bed Sensor Mat – RJ11
MED-B819-RJ11	Under Mattress Sensor Mat – RJ11



ABOUT THE DELUXE ALARM FALL MONITOR:


WHATS IN THE KIT:

- Monitor
- NC Connection Lead **(Only compatible with Tip & Sleeve Wiring)**
- Wall Mount Bracket
- Power Pack
- Silicon Strap

PROGRAMMABLE DEVICES: Can be programmed with up to 8 of the following devices:


- Under Mattress Bed Mats
- Above Mattress Bed Mats
- Chair Pads

MONITOR POWER: The Deluxe Alarm Fall Monitor is powered by 3 x AA batteries and/or a 6V power pack. When using both methods the batteries become a back-up power source.

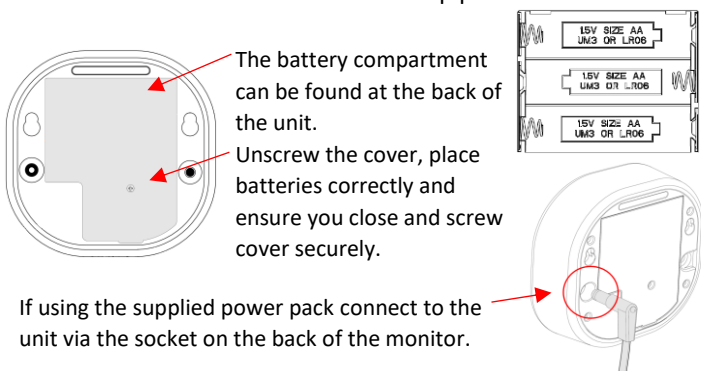

If using the supplied power pack connect to the unit via the socket on the back of the monitor.

NURSE CALL CONNECTION: Only Available to Nurse Call Systems with a Tip & Sleeve Wiring Configuration

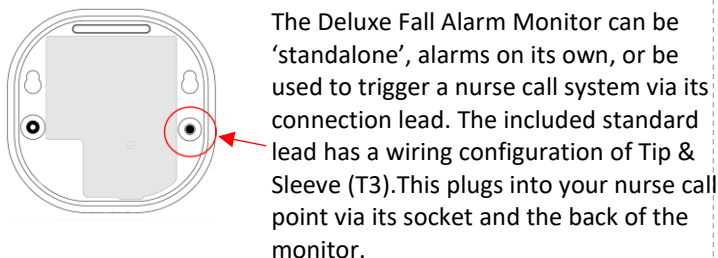

The Deluxe Fall Alarm Monitor can be 'standalone', alarms on its own, or be used to trigger a nurse call system via its connection lead. The included standard lead has a wiring configuration of Tip & Sleeve (T3). This plugs into your nurse call point via its socket and the back of the monitor.

Standard lead included is wired for Tip & Sleeve only.

Settings:


The settings can be found in the battery compartment at the back of the monitor.



NURSE CALL OUTPUT: When using the included connection lead to connect to your nurse call system the Nurse Call switch must be in the NO (normally open) position.




DELAY SETTINGS: Choose from 3 delay time options for bed/chair pads: 0sec, 2 sec or 15min.

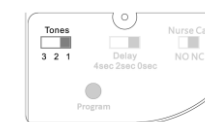

ALARM TONE: move the switch to choose the desired alarm sound. Choose from tones 1, 2 or 3.

Display:

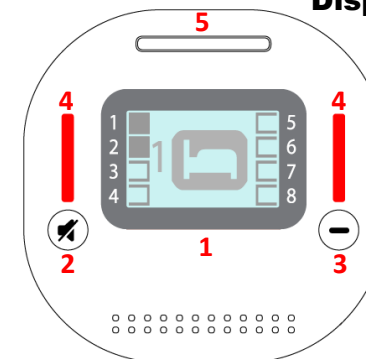

1. Display screen
2. Volume control/Mute button
3. Reset/Pause switch
4. LED visual alarms
5. To connect bracket or silicon strap


Mat/Pad Transmitter:

Mat/Pad RJ socket connection



The mat/pad transmitters are powered by 2 x AAA replaceable batteries. To replace the batteries, remove the 4 screws and back plate from the transmitter, carefully remove and replace the batteries. Replace the lid and screws. Connect a bed, under mattress or chair pad via its RJ plug into the socket and switch the on/off switch to the ON position to program or to set up programmed kit.

BED PAD POSITION

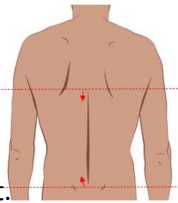

Placement of the bed pad is important to ensure it works correctly. Place pad on the solid base of the bed and under the mattress across the width of the bed. It should sit so that when the person is lying in bed it is no higher than their shoulder blades and not below their lower back. Ensure the transmitter pocket is not bent back over the pad, is down the side of the and will not be damaged by bed movement.

PROGRAMMING

To begin programming up to 8 wireless devices first ensure monitor is powered and all pads or mats to be programmed have the transmitter connected and in the on position.



Press and hold the program button until the monitor emits a 'beep' sound. Then trigger the device you wish to program.

You will hear another beep and the display will show the device is paired.



To jump positions to another number short press (tap) the program button until you get to the desired number.

To exit program mode, press the Reset button or the monitor will automatically exit after 10 seconds without operation.

Clear Programmed Devices:


To clear a programmed device, press the program button until the monitor emits a beep to denote it is in program mode. Tap the program button till the correct number of the device you wish to clear is shown. Press and hold the program button for 3 seconds until the icon disappears. Exit program mode as per above instructions.

Reset Alarm:

To reset an alarm short press the rest button. If there are numerous alarms showing and you wish to reset all press and hold the reset button until all cleared.

FUNCTIONS


Pause Function:

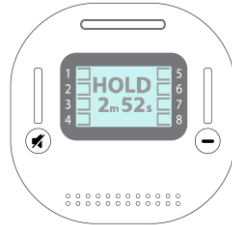

To pause the alarm on the monitor, press the Reset/Pause button for 3 seconds. The screen will display 'Hold' and the monitor will emit 3 beeps.

The display will also show the amount of time left before the pause function automatically switches off (10 minutes). To exit pause mode before the 10-minute period, press the Reset/Pause button. The monitor will emit a 'beep' and the LCD displays 'Hold' flashing 3 times and then disappears.

Volume Control & Mute Function:

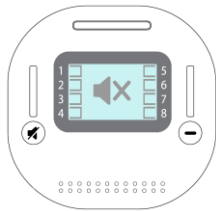


To choose volume short press the Volume control button until you get to the desired level: High, Medium or Low.

To mute, press and hold the volume control button until this display is shown. To exit mute mode, press the reset button.

Alarming Monitor Visual Screen:

Below is a description regarding what is happening on the pictured screen:

- All 8 slots have devices programmed.
- Devices 1 & 2 are alarming.
 - Slots are coloured in
 - LED indicator lights are flashing.
 - Picture changes to show devices triggered.

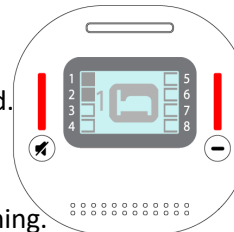


For more information regarding the icons shown on the screen please view page 6


ICON'S



WELCOME will be display when the monitor has no programmed devices.

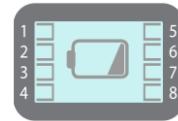

LOW BATTERY WARNING This icon will be displayed to show low batteries in the monitor. An alarm will sound on a 4 second cycle until the batteries are replaced.

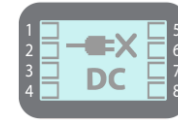

PLUG PACK DISCONNECT This warning will be generated if the power is unplugged or turned off. Re-connect power or if wanting to use batteries press the reset button to cancel alarm.



NURSE CALL DISCONNECT This will be display when the nurse call connection lead has been disconnected. Re-connect lead or if not required press the reset button.



TRANSMITTER OFFLINE WARNING If the monitor cannot find the transmitter for example it being out of range it will show this icon on its display.



Bed/Chair Alert



Floor Mat Alert



Call Button Alert



Paired Device



Paired Device-Transmitter Low Power



Bed/Chair Pad In Use



Bed/Chair Pad In Use- Transmitter Low Power



Alarming Device



Alarming Device- Transmitter Low Power

View our website:

www.indigocare.com.au

Contact us:

03-9763-2988

sales@indigocare.com.au