REDUCE THE RISK OF FALLS & INJURIES

LOW PROFILE, NON-SLIP MATERIAL

**PORTABLE** 

**EASY TO USE** 

EASY TO MOVE FROM ROOM TO ROOM

USE SOLVENT WIPES TO DISINFECTANT & CLEAN

INTEGRATES TO MOST NURSE CALL SYSTEMS

PLUG & PLAY OPERATION

**DURABLE & RELIABLE** 

COST EFFECTIVE

SMALL PRESSURE MAT (380mm X 255mm)



## Chair Sensor Pad – Corded RJ11 & JS63



(TGA Registration: 223902) – RCM Approved – AS/NZS61000.6.3.2012)

Normally Open system - MED-C365-NC10 (with RJ11) – requires Monitor and Y-Lead Normally Close System - MED-C365-JS63 (with 6.35mm Jack) – requires Interfacing Lead

Indigo Care provides a pressure sensitive Premium Chair Sensor Pad that is designed to immediately alert staff should a resident leave their chair. This is most useful to residents who are at risk of falling out of their chair, or those who require assistance in arising from the chair. The pad is placed in the centre of the chair, directly underneath the resident's bottom. A light covering can be placed on top of the chair sensor pad. However, if using a cushion, please test to ensure the sensor pad is working properly. Some types of memory foam cushions can affect the response time.

For a normally open call system, a monitor is required. Choose between our anti-tamper Deluxe Alarm Monitor (MED-TS1011A) or our Deluxe Voice Alert Monitor (MED-TS0893). The monitor is connected to the chair pad (MED-C365-NC10) and to the call point with a Y-cable. For a normally close call system, the chair pad (MED-C365-JS63) is connected directly to call point with an interfacing lead. Set-up instructions are printed on the actual mat.

## **KEY SPECIFICATIONS:**

- Dimensions: 380mm X 255mm
- Normally Open Connection: RJ11
   (telephone connector) on
   500mm curly cord. Requires an
   Anti-tamper Monitor or a Voice
   Alert Monitor & a Y-Lead
- Normally Close Connection:
   6.3mm jack plug. Requires
   Interfacing Lead
- Silver Printing, Resistance <1k</li>
   Ohm
- White PVC, Thickness 0.20mm
- Warranty: 12 months

## PRODUCT ACCESSORIES:

- RJ11 requires Y Cable
- Jack 6.3 requires straight 6.3mm or 3.5mm cable
- Voice Alert Monitor and Anti-Tamper Monitor require 4 X AA batteries-not included
- 6V DC Adaptor (optional for use with the Anti-Tamper)

ORDER INFORMATION
Email your purchase orders to sales@indigocare.com.au

INDIGO CARE (Horizon Electronics Company, T/as) 2/93 Rushdale Street, Knoxfield, VIC, 3180 PH +61 3 9763-2988 www.indigocare.com.au

ABN 36007302801