

# Bed Sensor Mat – Corded RJ11 & JS63

REDUCE THE RISK OF  
FALLS & INJURIES

LOW PROFILE,  
NON-SLIP MATERIAL

PORTABLE

EASY TO USE

EASY TO MOVE FROM  
ROOM TO ROOM

USE SOLVENT WIPES  
TO DISINFECTANT &  
CLEAN

INTEGRATES TO MOST  
NURSE CALL SYSTEMS

PLUG & PLAY  
OPERATION

DURABLE & RELIABLE

COST EFFECTIVE

MORE COMFORT  
LESS NOISE

MEDIUM SIZED  
PRESSURE MAT  
(770mm X 495mm)



(TGA Registration: 223902 – RCM Approved – AS/NZS61000.6.3.2012)

Normally Open system -MED-B365-NC20 (with RJ11) – requires Monitor & Y-Lead  
Normally Close system- MED-B365-JS63 (with 6.3mm Jack) – requires Interfacing Lead

Indigo Care provides an improved pressure sensitive Premium Bed Sensor Mat that is more comfortable and less noisy for the user. The mat's thickness has increased to 12mm providing a softer touch and feel for the user at the same time ensuring the noise created due to contact is quieter.

Our mats are designed to immediately alert staff should a resident leave their bed or for those who require assistance in arising from the bed. The mat is placed on the bed, directly under the bed linen. For a normally open call system, a monitor is required. Choose between our anti-tamper Deluxe Alarm Monitor (MED-TS1011A) or our Deluxe Voice Alert Monitor (MED-TS0893). The monitor is connected to the bed mat (MED-B365-NC20) and to the call point with a Y-cable. For a normally close call system, the bed mat (MED-B365-JS63) is connected directly to call point with an interfacing lead. Set-up instructions are printed on the actual mat. To prevent the bed mat from moving, you can tie it down using the eyelet holes on the mat.

## KEY SPECIFICATIONS:

- **Mat Dimensions: 770mm X 495mm**
- **Normally Open Connection: RJ11 (telephone connector) on 500mm curly cord. Requires an Anti-tamper Monitor or a Voice Alert Monitor & a Y-Lead**
- **Normally Close Connection: 6.3mm jack plug. Requires Interfacing Lead**
- **Electrical resistance < 500 Ohm**
- **White PVC, Thickness 0.20mm**
- **Warranty: 12 months**

## PRODUCT ACCESSORIES:

- **RJ11 requires Y Cable**
- **Jack 6.3 requires straight 6.3mm or 3.5mm cable**
- **Voice Alert Monitor and Anti-Tamper Monitor require 4 X AA batteries-not included**
- **6V DC Adaptor (optional for use with the Anti-Tamper)**

## ORDER INFORMATION

Email your purchase orders to  
[sales@indigocare.com.au](mailto:sales@indigocare.com.au)

**INDIGO CARE**  
(Horizon Electronics Company, T/as)  
2/93 Rushdale Street, Knoxfield, VIC, 3180  
PH +61 3 9763-2988  
[www.indigocare.com.au](http://www.indigocare.com.au)  
ABN 36007302801